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All instruments have been developed and constructed for their specific use. Improper use can lead to damage to the tissue, premature wear, destruction of the instruments, danger to the user, patient or third parties.

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- Use of improper instruments can cause insufficient preparation results.
- Excessive speeds while using instruments with overly large heads endanger the stability of the instruments.
- Instruments must be rotating before contact is made.
- Observe permissible speeds indicated on packaging.
- Make sure that only technically and perfectly cleaned turbines, handpieces and contraangles are used.
- Use a suction device in the laboratory.
- Improper use leads to increased risk and inferior results.
- The recommended applied contact pressure range is 0.3 – 2 N (30 – 200 p) depending on the kind of preparation.
- Avoid tilting, levering, or blocking the instruments up by using too much contact pressure (increased danger of fracture).
- Insert the instruments as deeply as possible without excessive use of force into the chuck and check to ensure that they are secure.
- Extremely tapered instruments will wear faster at the tip due to the smaller size. To maintain the longevity of these instruments, careful preparation with less contact pressure is important.
- Apply Turbo instruments over their entire grinding surface.
- Roughness obtained with a diamond coating >ISO 524 has to be finished.
- Immediately remove any instruments that are damaged, bent or no longer run concentrically.
- Depending on the application, the use of protective goggles is recommended.
- The use of inappropriate disinfectants and/or keeping the instruments too long in the disinfectant can result in corrosion (increased danger of breakage) and discolouration of the instruments or the colour coding can come off.

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- No alkaline disinfectant must be used for the disinfection of silicon rubber polishers.
- Do not place polishers in alcohol!
- Use brushes / Flexi-Snap as single-use products! (X)

Observe the instructions for use
www.edenta.com

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Storage and Keeping Conditions:

- In dry conditions and protected against contaminants. Protect instruments in general against chemicals, acids, heat and extreme temperature variations. Do not use polishers after expiration date indicated.
- **Steel instruments** are unsuitable for both mechanical preparation and the steam steriliser. They should be replaced by corresponding tungsten-carbide instruments.
- **Single-use products** (identified on the packaging with (X)) are not approved for reuse. Safe use cannot be guaranteed if these products are reused, as there is the risk of infection and/or the products are no longer safe to use.
- The end of the product's service life is generally determined by wear and damage due to use.

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Recommendations for use

Polishers / Brushes

- Apply low contact pressure in order to minimize heat generation.
- Polish with circling movements.
- In order to achieve a high-shine polish, for multiple step polishing procedures all polishers are to be used in the indicated sequence.
- Eye protection is recommended.

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Sintered diamonds

For cleaning, have to be dressed with the cleaning stone Art. No. 9920 from time to time:

- Water cleaning stone to minimize dust formation and to prevent damage to the abrasive.
- Short-time grinding on the cleaning stone at 8.000 rpm
- To exclude any damage to the instrument shank, avoid contact of the cleaning stone with the shank during the grinding procedure.

Sharpening of DSB abrasives:

- Sharpening of blunt Sintered diamonds is carried out in the same way as the cleaning procedure but for a longer period of time. Thus, the bonding material containing used, blunt grains is removed and new, sharp grains are exposed.

Cleaning of cleaning stone

- It is recommended to subsequently clean the cleaning stone in the ultrasonic bath during 1 - 2 minutes.

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Contact pressure

Excessive contact pressure has to be avoided because this can lead to damage and breakouts on the working parts of bladed instruments. Moreover, increased heat generation occurs.

- Increased contact pressure may lead to stripping of the grit on abrasive instruments and increased heat generation.
- Increased operating pressure may also lead in case of breakouts on the blades to undesirably rough surfaces. Instrument breakage may occur.

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Cooling for FG diamond instruments

- In order to avoid undesirably high heat generation, sufficient air / water spray (50 ml/min at minimum) has to be provided.
- For FG instruments with a total length of more than 22 mm or a head diameter of more than 2 mm, additional external cooling is necessary.
- Insufficient water cooling can cause irreversible damage to the materials and the instruments.

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